




## INSTALLATION AND OPERATING INSTRUCTIONS

### INSTALLATION:

1. Open the trunk or tailgate. Set the bottom of the Lift frame on the bumper and the top on the ground.



2. Loop the cord on the bottom of the Lift through the lock latch in the trunk. Then thread the cord through the clamp with the thumbscrew on it. Leave it loose enough to allow you to close the trunk.

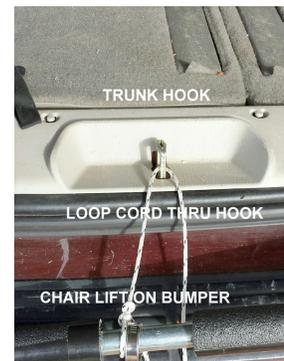

3. **NOTE:** If you cannot loop the cord from the bottom of the Lift frame through the trunk latch, use the cylindrical retainer shown in the adjacent photo. Place the retainer in the trunk with the cord extending outside. Close the trunk lid and pull the cord tight.



4. Close the trunk lid. If there is a problem closing it securely, adjust the position of the cord and try again.

5. Tip the Lift up, and hook the two top straps on the trunk lid. Adjust them so that the Lift leans slightly toward you as shown.



6. Check that the bottom of the Lift frame is positioned on the bumper. Then pull the cord until the bottom of the Lift frame is securely positioned against the trunk lid. Tighten the thumbscrew to lock the cord in place.
7. Pull on the Lift to verify that the cord and top straps are tight.

### **LOADING THE WHEELCHAIR:**

8. Roll the wheelchair backwards up against the vehicle's bumper.

9. Place the lifting bar under the wheelchair's push handles and hold it in place until you have started lifting the chair.



10. Turn the crank clockwise to lift the chair until the handles are secured between the lift bar and the axle shaft.



11. Add the safety strap and check to make sure everything is securely in place.

### **UNLOADING THE WHEELCHAIR:**

12. Remove the safety strap.
13. Turn the crank counterclockwise to lower the wheelchair and remove the lifting bar from under the wheelchair push handles.

### **STORING THE WHEELCHAIR LIFT IN THE TRUNK:**

14. Loosen the thumbscrew locking the cord attached to the bottom of the Lift frame and loosen the cord. Pull the lift toward you, away from the trunk far enough that the trunk lid has room to open.
15. Remove the Lift's top straps from the trunk lid.
16. Tip the Lift back to rest on the ground.
17. Open the trunk, tip the Lift inside and close the trunk.



**CAUTION: Remember to check and tighten the cords and straps each time the Lift is used. Check them frequently when transporting your wheelchair.**